

SensPERT™

Feline Infectious Peritonitis Test Kit



Feline Infectious Peritonitis Test Kit



Summary : Detection of anti-Feline Infectious Peritonitis Virus N protein antibody from feline whole blood, serum, or plasma within 10 minutes

Principle : One-step immunochromatographic assay

Detection Target : Feline Coronavirus Antibody

Sample : Feline Whole Blood, Plasma, and Serum

Reading time : 5 ~ 10 minutes

Sensitivity : 98.3% vs IFA

Specificity : 98.9 % vs IFA

Contents : Test kit, Buffer bottle, and Disposable droppers

Storage : Room Temperature (at 2 ~ 30 °C)

Expiration : 24 months after manufacturing

Caution :

Use within 10 minutes after opening

Use appropriate amount of sample (1 drop(10 μ l) of a dropper)

Use after 15~30 minutes at RT if they are stored under cold circumstances

Consider the test results as invalid after 10 minutes


Instruction for FIP Ab test



Sample fill line (black line) : 1 drop (10 μ l)

1


Sampling & loading

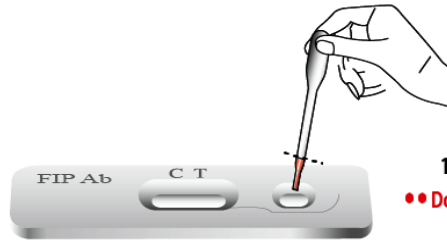

1 drop of sample
•• Do not overload the sample !!

2

Absorption

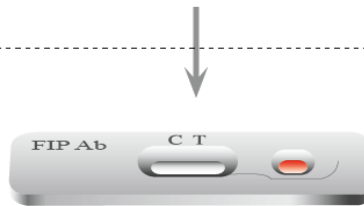

Wait for complete absorption

3

Buffer addition



2 drops of buffer
•• after complete absorption



4

Reading

00:05:00~00:10:00



Read the result between 5 ~ 10 min



5

Discard



Discard the used device
In valid result after 10 min

