

SensPERT™  
Feline Toxoplasma IgG/IgM Ab Test Kit



# Feline Toxoplasma IgG/IgM Ab Test Kit



**Summary :** Detection of anti-Toxoplasma IgG/IgM antibody from feline whole blood, serum, or plasma within 10 minutes

**Principle :** One-step immunochromatographic assay

**Detection Targets :** Anti-Toxoplasma IgG/IgM antibody

**Sample :** Feline Whole Blood, Plasma, and Serum

**Reading time :** 5 ~ 10 minutes

**Sensitivity :** 98.0% vs IFA for IgG / 97.4% vs IFA for IgM

**Specificity :** 98.9 % vs IFA for IgG / 98.2% vs IFA for IgM

**Quantity :** 1 box (kit) = 10 devices (Individual packing)

**Contents :** Test kit, Buffer bottle, and Disposable droppers

**Storage :** Room Temperature (at 2 ~ 30 °C)

**Expiration :** 24 months after manufacturing

**Caution :**

Use within 10 minutes after opening

Use appropriate amount of sample (0.01 ml of a dropper for Toxo IgG/IgM)

Use after 15~30 minutes at RT if they are stored under cold circumstances

Consider the test results as invalid after 10 minutes


# Instruction for Feline Toxo IgG /IgM Ab test



Sample fill line (black line) : 1 drop ( 10  $\mu$ l )

# 1


Sampling  
& loading

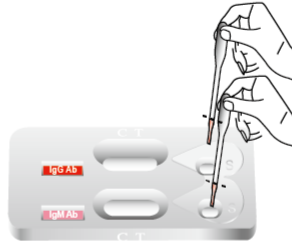

1 drop of sample  
•• Do not overload the sample !!

# 2

Absorption

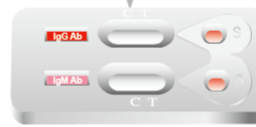

Wait for complete absorption

# 3

Buffer addition

00:00:00



2 drops of buffer  
•• after complete absorption



# 4

Reading

00:05:00~00:10:00




+



Read the result between 5 ~ 10 min

-

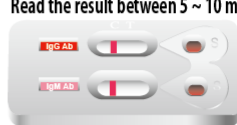

# 5

Discard

00:10:00



Discard the used device  
In valid result after 10 min

