

SensPERT™
Canine Distemper Virus Test Kit



Canine Distemper Virus Test Kit



Summary : Detection of specific antigens of canine distemper virus within 10 minutes

Principle : One-step immunochromatographic assay

Detection Targets : Canine Distemper Virus (CDV) antigens

Sample : Canine ocular discharge and nasal discharge

Reading time : 5 ~ 10 minutes

Sensitivity : 98.6% vs RT-PCR

Specificity : 100.0 % RT-PCR

Quantity : 1 box (kit) = 10 devices (Individual packing)

Contents : Test kit, Tubes, Disposable droppers, and Cotton swabs

Storage : Room Temperature (at 2 ~ 30 °C)

Expiration : 24 months after manufacturing

Caution :


Use within 10 minutes after opening

Use appropriate amount of sample (0.1 ml of a dropper)


Use after 15~30 minutes at RT if they are stored under cold circumstances

Consider the test results as invalid after 10 minutes


Instruction for CDV test

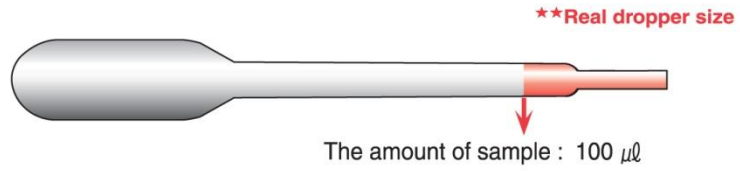

1 Sampling

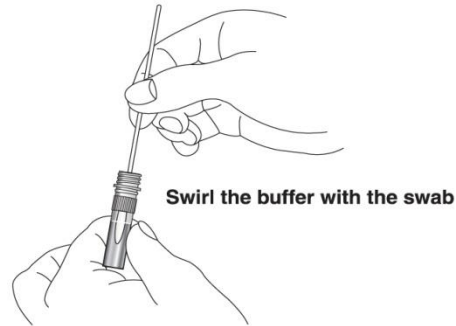

2 Loading

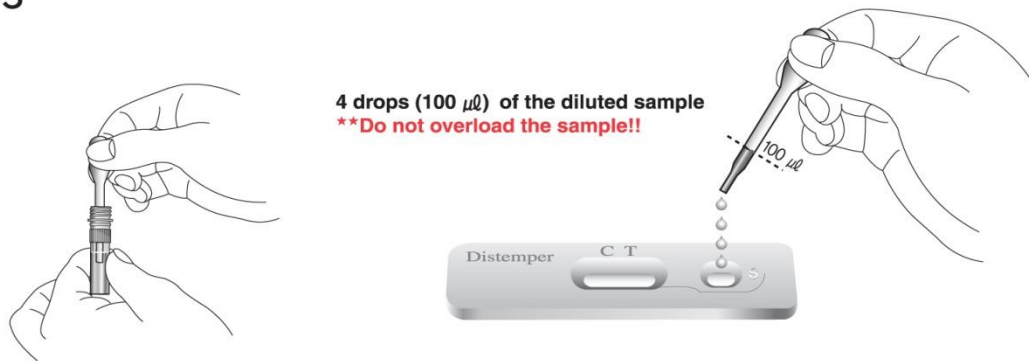

3 Reading

00:05:00~00:10:00



Read the result between 5~10 min



4 Discard

00:10:00



: Discard the used device
Invalid result after 10 min

